



Heavenly Use Guide

Prepare the Cloud

Lift the lever upward to unlock and allow filling.

Set Your Intention

Turn the dial to choose your dose. Each click marks a precise interval.

Seal the Setting

Press the lever down to lock your selected dose in place.

Release

Dispense with smooth, steady pressure and audible clicks.

Designed for controlled flow, smooth output, and heavenly precision.

CloudDose → Tox Units Chart

CloudDose Marking (mL)	Tox Units
0.1 mL	4 units
0.2 mL	8 units
0.3 mL	12 units
0.4 mL	16 units
0.5 mL	20 units
0.6 mL	24 units
0.7 mL	28 units
0.8 mL	32 units
0.9 mL	36 units
1.0 mL	40 units

Overview of tox units on a syringe based on the standard 100 units reconstituted with 2.5 mL saline.

CloudDose was designed for precision, consistency, and control when working with tox.

Each 1 mL syringe features an adjustable dial that allows you to set your intended number of units. As you dispense, the syringe clicks when you reach that amount so you know exactly how much you're using, every time.

CloudDose is made from higher-end plastic compared to standard syringes, offering added durability and a more controlled feel during use.

Use once, then discard.

If you choose otherwise, soak in 70% isopropyl alcohol for 10 minutes, similar to how many reuse MN cartridges.



CLOUD ELEVEN

CloudDose Tox Click Chart 100 Unit Vial Reconstituted with 2.5 mL 40 Units per 1 mL

Setting (mL)	Units per Click	1 Unit	2 Units	4 Units
0.01	0.4	2–3 clicks	5 clicks	10 clicks
0.02	0.8	1–2 clicks	2–3 clicks	5 clicks
0.04	1.6	1 click	1–2 clicks	2–3 clicks

Note: This chart also applies to Innotox

CloudDose Tox Click Chart (1:1 Ratio) 100 Unit Vial Reconstituted with 1.0 mL 100 Units per 1 mL

Setting (mL)	Units per	1 Unit	2 Units	4 Units
0.01	1	1 click	2 clicks	4 clicks
0.02	2	1 click	1 click	2 clicks
0.04	4	1 click	1 click	1 click

Overview of tox units on a syringe based on the standard 100 units reconstituted with 2.5 mL saline.