

PROTOCOL SUGGESTION: HAIR MESOTHERAPY

Point By Point Application	Nappage
Depth: 2-2,5mm	Depth: 0,5-1,5mm
Freq: 0-60	Freq: 300-1500
Press: Starting with 20%	Press: 10% - 20 %

Press adjustment should be identified by professional ; based on skin properties, products viscosity and the needle size



PROTOCOL SUGGESTION: FACE / NECK MESOTHERAPY

Point By Point Application	Nappage
Depth: 2-2,5mm	Depth: 0,5-1,5mm
Freq: 0-60	Freq: 100-700
Press: Starting with 20%	Press: Starting with 10%

Press adjustment should be identified by professional; based on skin properties, products viscosity and the needle size



PROTOCOL SUGGESTION: PERIORBITAL AREA MESOTHERAPY

Point By Point Application	Nappage
Depth: 0,5-1 mm	Depth: 0,5-0,7mm
Freq: 0-20	Freq: 100-200
Press: Starting with 10%	Press: Starting with 5%

Press adjustment should be identified by professional; based on skin properties, products viscosity and the needle size



PROTOCOL SUGGESTION: HIPERPIGMENTATION (SKIN SPOT)

Point By Point Application

Depth: 1-3mm

Freq: 0-20

Press: Starting with 10%

Depth adjustment should be done based on the pigmented area.

Press adjustment should be identified by professional; based on skin properties, products viscosity and the needle size



PROTOCOL SUGGESTION: LIPOLYSIS /CELLULITE TREATMENT

Point By Point Application


Depth: 4 – 10 mm

Freq: 0-20

Press: Starting with 20%

0,1-0,6ml lipolytic product injection in each point is suggested for the targeted treatment.

Press adjustment should be identified by professional; based on skin properties, products viscosity and the needle size

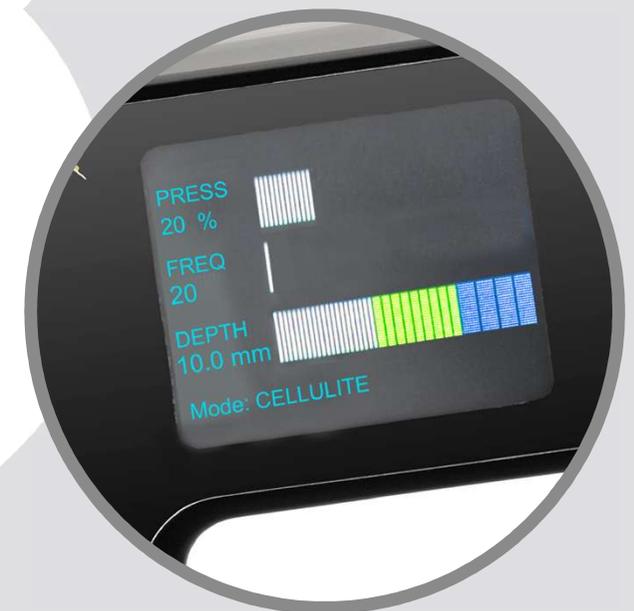

PROTOCOL SUGGESTION: SCAR TREATMENT

Pen Mode Application	Point By Point Application	Nappage
Depth: 0,5 – 2 mm	Depth: 2-3mm	Depth: 1-1,5mm
Freq: 1200 - 1500	Freq: 20-60	Freq: 300-700
Press: Starting with 5%	Press: Starting with 20%	Press: Starting with 10%

Press adjustment should be identified by professional; based on skin properties, products viscosity and the needle size

