

Trichomonas vaginalis/Candida albicans/Gardnerella vaginalis 3in1 Antigen Rapid test

The Trichomonas vaginalis / Candida albicans / Gardnerella vaginalis 3-in-1 Panel STD Rapid Test Kit is a lateral flow immunoassay for the qualitative detection of antigens from Trichomonas vaginalis, Candida albicans, and Gardnerella vaginalis.

Format: 20Tests

Sample: 20 sterilized Swabs

Sample buffer: 20 vials

Results: in 15 minutes

Storage Temperature: 2-30 °C

Available from our German warehouse – No long lead times

To Get a Quote:

hello@reinventhealthcare.eu




✓ Easy Sampling

✓ Quick Results

✓ >90% Accuracy

✓ No extra equipments are needed

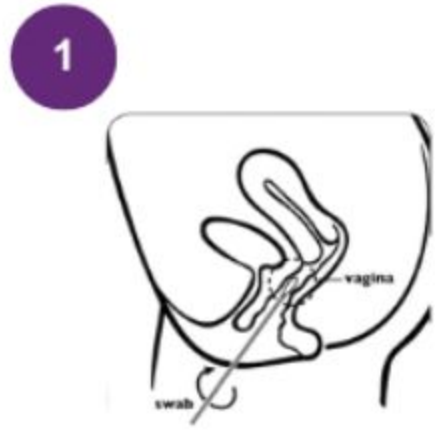


1 Prepare the collected sample. Insert the swab until the fingers reach the outer labia, then rotate 5 times to collect the sample.



2 Open the sample buffer tube.



3 Place the swab into the tube and rotate it several times in the sample buffer.

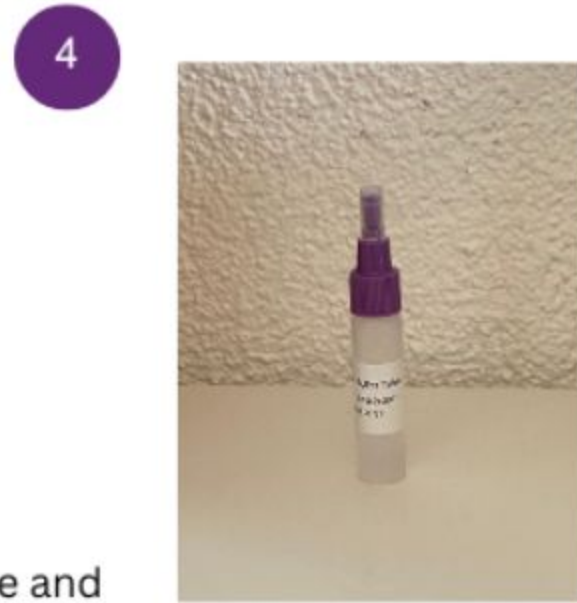

4 Remove the swab from the tube and securely close the cap.



5 Probenpuffer 30 Sekunden gleichmäßig schütteln.



6 Remove a JTCGV test cassette from the foil pouch.



7 Remove the tip cap from the extraction tube.



8 Pipette 3–4 drops of the sample solution (approx. 80 μ L) into the sample well ('S') of the test device.



9 Read the test result after 15 minutes.

