

## Trichomonas vaginalis/Candida albicans/Gardnerella vaginalis 3in1 Antigen Rapid test

The Trichomonas vaginalis / Candida albicans / Gardnerella vaginalis 3-in-1 Panel STD Rapid Test Kit is a lateral flow immunoassay for the qualitative detection of antigens from Trichomonas vaginalis, Candida albicans, and Gardnerella vaginalis.

Format: 20Tests

Dropper: 20 pcs

Sample buffer: 20 vials

Results: in 15 minutes

Storage Temperature: 2-30 °C

**Available from our German warehouse – No long lead times**

**To Get a Quote:**

**hello@reinventhealthcare.eu**



✓ Easy Sampling

✓ Quick Results

✓ >90% Accuracy

✓ No extra equipments are needed

1




Collect urine.

2



Öffnen Sie den Deckel des Probenpuffers.

3

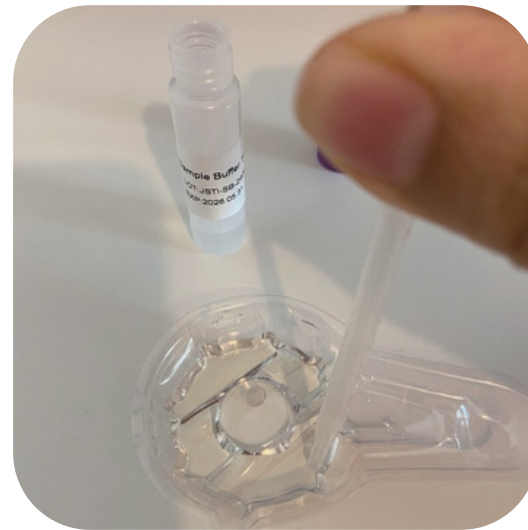

Pipette 0.5mL Urine to the Sample Buffer tube.

4



Close the Sample Buffer Tube well. Do not let the sample buffer overflow.

5



Swirl the sample buffer tube clockwise for total 5 times until the mixture is uniform.

6



Remove the test cassette from the foil pouch and open the bottom of the cassette cover.

7



Break the sample buffer tube cap.

8



Squeeze total 4 drops into each sample hole.(S) region on the test cassette.

9




Read the result at 15 minutes. Do not read the result earlier than 15 minutes or after 20 minutes.

10

Read the results

**POSITIVE**

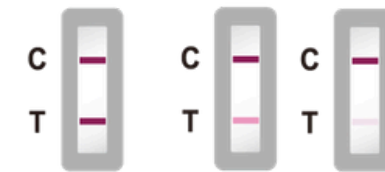

**NEGATIVE**



**INVALID**

