AutoMini Digital

DANSK: Personlig slyngeforstærker til tv'et	side 2
ENGLISH: Personal loop amplifier for TV	page 13
FRANÇAIS: Amplificateur en boucle personnel pour télévision	page 24
NEDERLANDS: Persoonlijke ringleidingversterker voor TV.....	Pag. 35
NORSK: Personlig slyngeforsterker til TV-apparatet	side 46
SVENSKA: Personlig slingförstärkare till TV:n	sid 57

Ordering number: 300 00 06
AutoMini Digital Set, EU

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Practical and simple loop amplifier

AutoMini is a personal amplifier for people with a hearing instrument. It amplifies and transmits sounds from, for example, a TV. The amplifier is easy to use and provides several connection options, both for picking up TV sound and listening. When it has been connected, you do not need to do anything when listening to the TV, the amplifier is always on. Set the hearing instrument to the T or MT position.

Because of its convenient shape and size and simple connection, it is easy to take the amplifier with you. This is practical when you want to be able to listen to TV, e.g. in a summer cottage, or at friends or in a hotel room. Therefore make sure you keep this user manual and all the cords supplied!

Intended purpose:

The intended purpose of Automini Digital is to enhance the possibility for hard of hearing people to hear audio information.

Intended user:

The intended user is a person with mild to severe hearing loss who needs audio with better signal to noise ratio.

Intended user group:


The intended user group consists of people of all ages with mild to severe hearing loss using hearing aids or cochlear implants that supports listening to hearing loops (via telecoil).

Principle of operation:

The Automini Digital converts the given audio signal to a magnetic field which then can be picked up by hearing aids or cochlear implants, with built in telecoils, in order to transfer it to the intended user.

System components. The following parts are supplied:

Amplifier and mains adapter

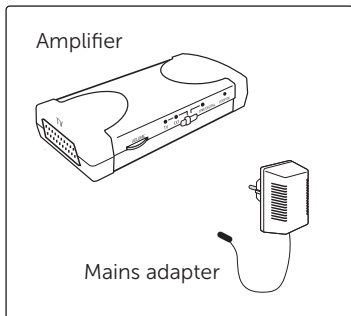For picking up TV sound

Option A1:
Optical TOSLINK -
mini TOSLINK cable

Option A2:
Coax. RCA-3.5mm
cable


Option B:
Line-out

Audio cable
3,5 mm


Listening loop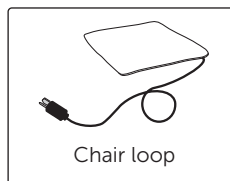

Connection


1. Connecting the mains adapter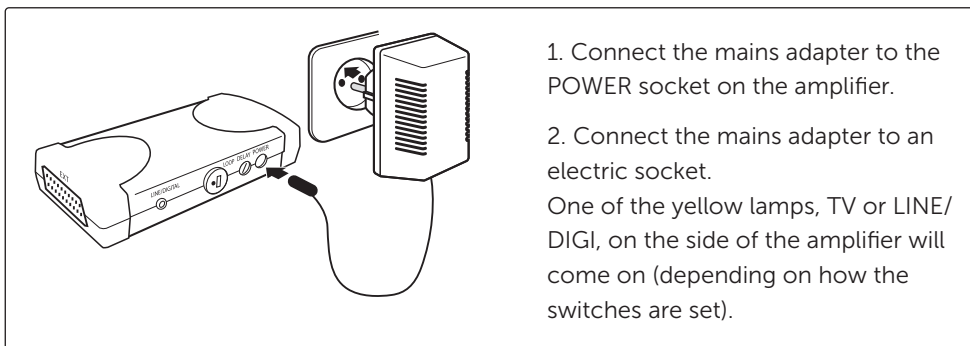

2. Picking up the TV sound

The amplifier should be connected to the TV. The amplifier works just as well if you already have one or more external units connected to the TV (e.g. video, DVD, digibox, satellite receiver). There are three different ways of transmitting the sound. For the best sound quality and simplest connection method, A is recommended as the preferred option, and then option B, C and then D.


Connection option A1: Digital, optical connection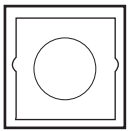


Check if your TV has a digital audio output. This socket is often marked with "Digital Output (Optical)", but it varies between different brands of TVs (check the TV's instruction manual).

--> If your TV has not a digital audio output, go to connection option B.

Connection option A2: Digital, Coax connection (extra accessory)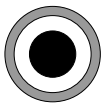

Check if your TV has a digital Coax output (typically labeled "Digital Out-put (Coax)"). Use a Coax. RCA-3.5 mm cable to connect to the TV (ordered separately).

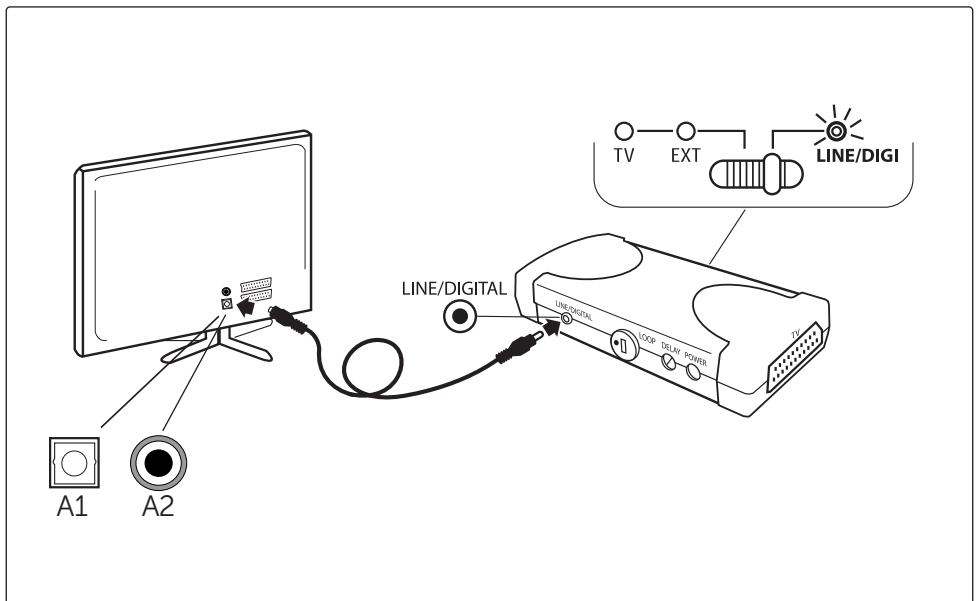
Connection option B: RCA Line out or 3.5 mm audio output jack


Check whether your TV has a Line Out output (usually on the rear of the TV, see TV user manual). Use the RCA Line out connector cable to connect the AutoMini:

--> If your TV does not have a Line Out output, move on to connection option C.

Please note! Some TVs have a sound input on the front or on the side which looks like a Line Out output but which is NOT able to transmit sound to an amplifier. This is usually marked "Audio" and is located next to sockets marked "Video" and "S-Video" (see TV user manual).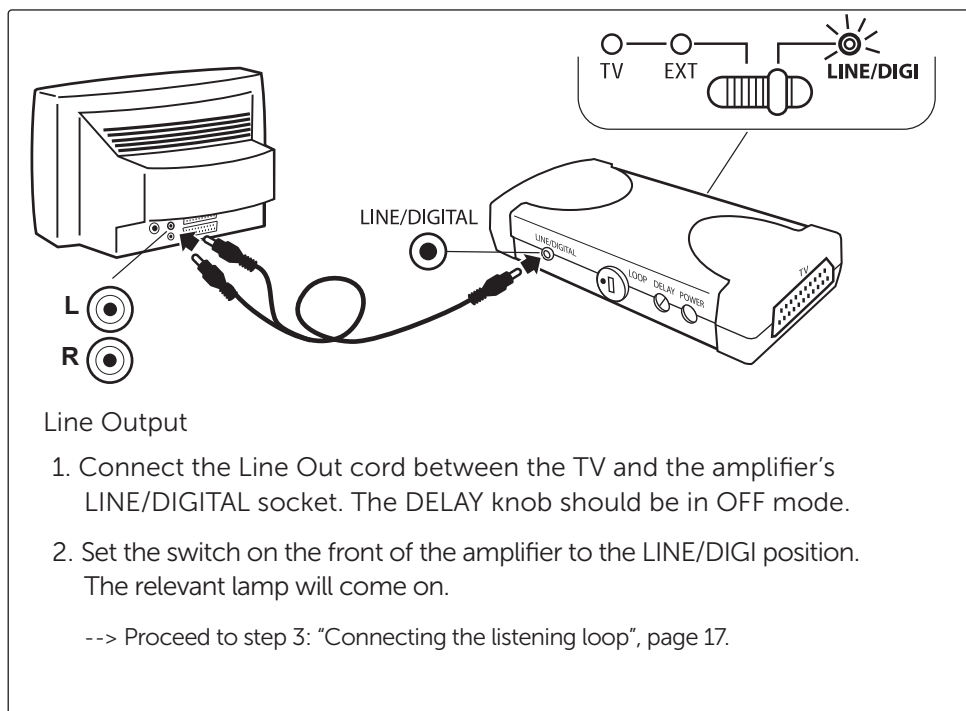


If your TV does not have a Line Out output connector check if it has a 3.5 mm audio output jack (can be located both on the front and the rear of the TV). If so, use the 3.5 mm audio cable to connect the AutoMini: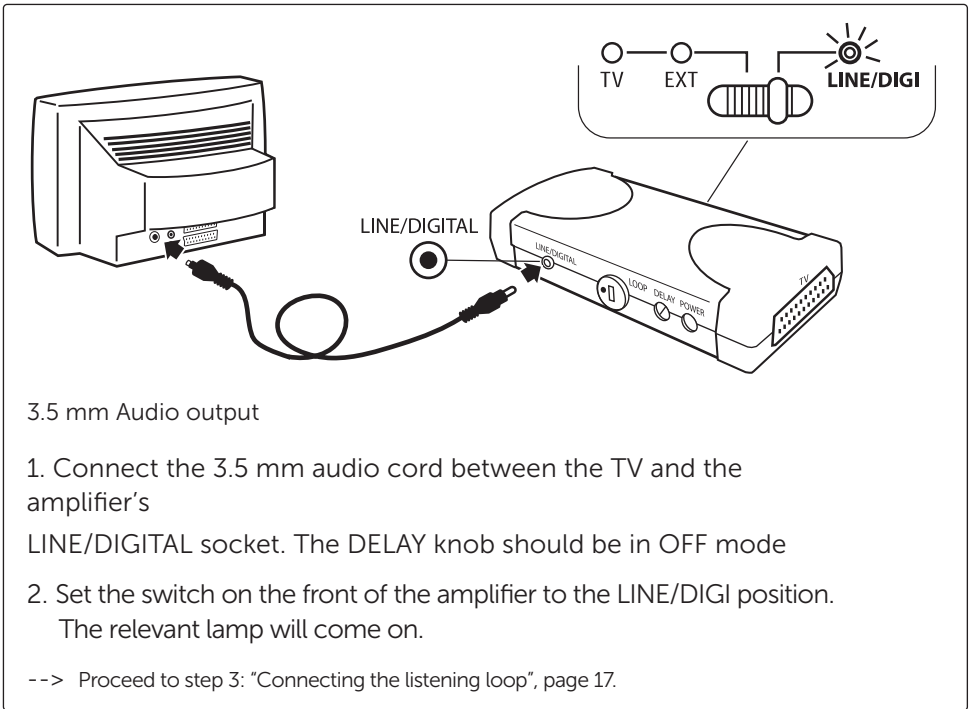

If your TV does not have a Line Out or a 3.5 mm audio output, move on to connection option C.


Please note! Some TVs have a sound input on the front or on the side which looks like a Line Out output but which is NOT able to transmit sound to an amplifier. This is usually marked "Audio" and is located next to sockets marked "Video" and "S-Video" (see TV user manual).

Connection option C: SCART connector (Extra accessory)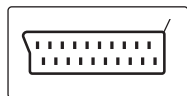

Check whether your TV has a SCART socket. It does not matter whether it is in use.

--> If it does not have one, proceed to connection option D.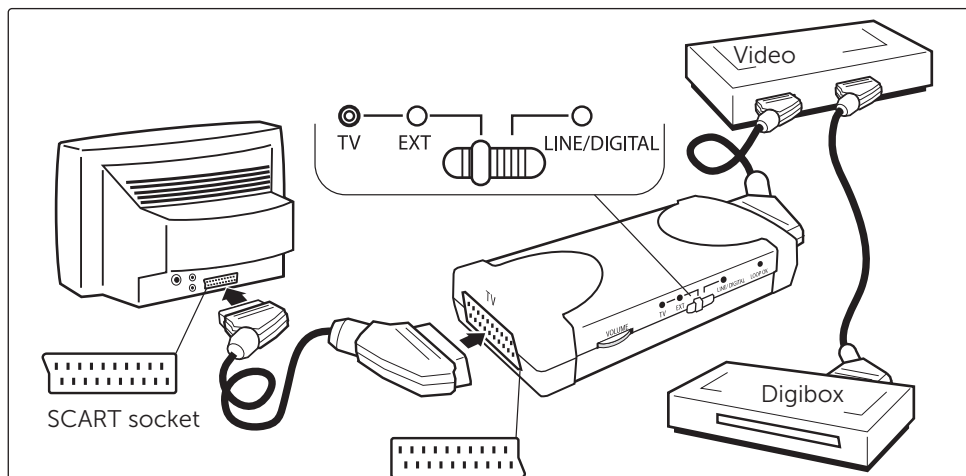

1. Connect the amplifier to the TV SCART socket using the SCART cord supplied. If the TV has more than one SCART socket, use the one that the TV manual recommends for video (usually SCART 1 or AV1).

If the TV's SCART socket is being used by one or more external units (e.g. video, DVD, digibox, satellite receiver), connect the amplifier between the TV and the external units. The units which are connected to the TV should be connected in series (see illustration above). The DELAY knob on the backside of AutoMini should be in OFF mode.


2. Set the switch on the front of the amplifier to the TV EXT position. One of the lamps, TV or EXT, will come on, depending on whether the TV or an external unit is being used at present.

--> Proceed to step 3: "Connecting the listening loop", page 15.

Please note! When correctly connected, the TV picture and sound will automatically be changed to the programme source you selected (e.g. video, DVD, digibox, satellite receiver). If you are watching video, DVD, satellite TV or another external unit and then switch to a standard analogue TV channel, in some cases you may, however, receive the wrong sound on the TV. In that case switch off the external unit, or all the external units, to obtain the correct sound. This phenomenon will never occur if you use connection options A, B or D.

Connection option D: Microphone (Extra accessory)

If the TV does not have an Optical, Line Out socket or SCART, use an FXT microphone (ordered separately) to pick up the TV sound.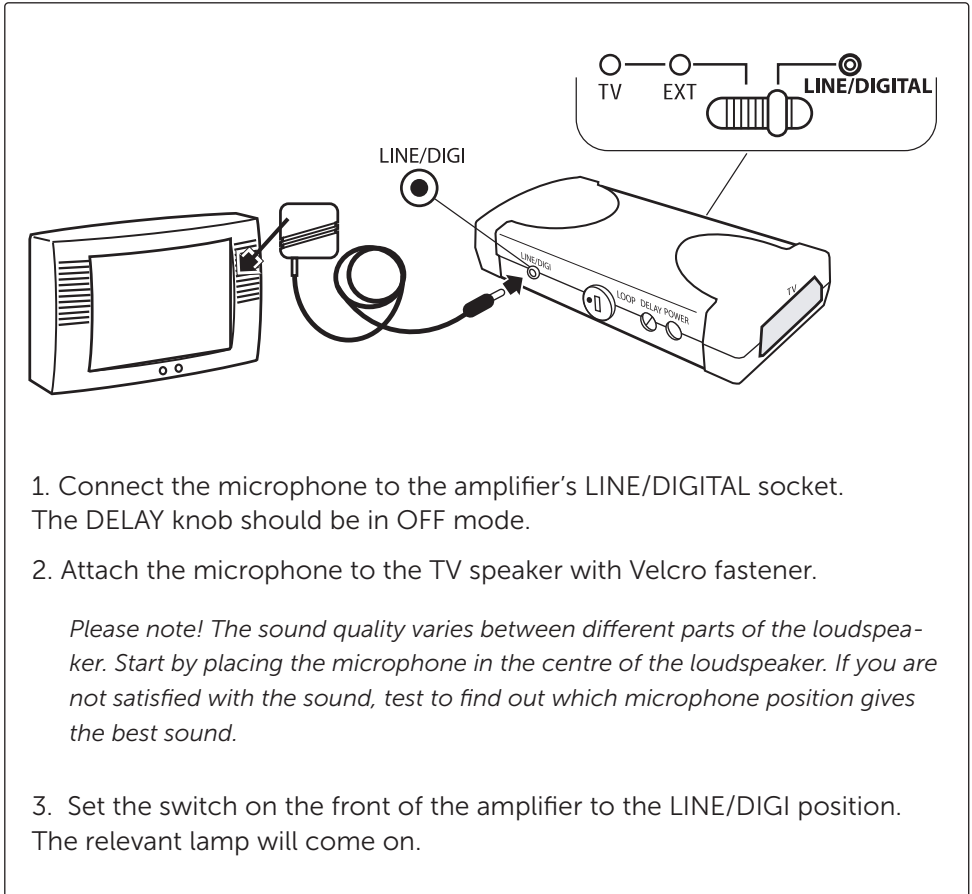

1. Connect the microphone to the amplifier's LINE/DIGITAL socket. The DELAY knob should be in OFF mode.
2. Attach the microphone to the TV speaker with Velcro fastener.


Please note! The sound quality varies between different parts of the loudspeaker. Start by placing the microphone in the centre of the loudspeaker. If you are not satisfied with the sound, test to find out which microphone position gives the best sound.

3. Set the switch on the front of the amplifier to the LINE/DIGI position. The relevant lamp will come on.

3. Connecting the listening loop

Select how you wish to listen to the sound. Wearers of hearing instruments can listen via the chair loop supplied or via our neck loop which can be purchased as an accessory.

Listening option A: Chair loop

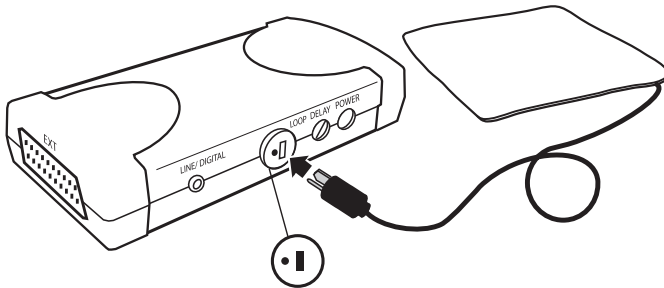1. Turn the volume down to minimum (0) so that the sound level is not too high at the outset. The DELAY knob should be in OFF mode.
2. Insert the chair loop connector in the amplifier's LOOP socket. The green LOOP OK lamp comes on.
3. Place the chair loop on the seat, either below or above the seat cushion.
4. Lay the cord supplied along the wall. Make sure that the cord is positioned so that you cannot trip up on it.
5. Set the hearing instrument to the T or MT position.
6. Adjust the volume control to a comfortable level and sit on the chair loop when listening.

Listening option B: Neck loop (accessory)

Our neck loop with volume control is an alternative to the chair loop. Sold separately. Fitted as set out in step 1 and 2 above and worn round the neck. Set the hearing instrument to the T or MT position.

Synchronization of sound - the DELAY knob

Unsynchronized sound can be found in certain LED, LCD or PLASMA TV sets. If the sound is not synchronized with the image (the sound precedes the image) you can adjust this with the knob marked "DELAY" on the backside of AutoMini. Increase the delay of the sound by turning the DELAY knob clockwise until you find that the sound is synchronized with the image. This should only be needed to do once.

4. Volume control

The amplifier has an automatic volume control (AGC = Automatic Gain Control), which means that the sound level in the loop remains constant, even if the sound level on the TV changes. If necessary, you can adjust the sound level using the volume control on the amplifier. You will normally only need to do this once.

Troubleshooting

Type of fault	Solution
There is no sound, or there is a humming sound:	<ol style="list-style-type: none"> 1. Check that the amplifier's mains cable is connected. 2. Check that the switch on the front of the amplifier is set correctly (see pages 11-14). 3. Test by moving the chair loop to another seat. (The hum may be due to electrical fields in the room.) 4. If you are using connection option B, Line Out, check that the socket is NOT a sound input for a video camera (see page 12 and/or the TV user manual). 5. Check your TV's audio settings (or advanced audio settings). Make sure that the PCM setting is activated. Please note that the AutoMini Digital does not support ie. Dolby.
The sound is faint:	<ol style="list-style-type: none"> 1. Increase the volume on the amplifier. 2. If you are using the microphone, check that it is attached to the TV speaker and that the TV volume is on. 3. Check that the hearing instrument is in the T or MT position. 4. Check that the chair loop is connected. Sit on the chair loop.
The LOOP OK lamp is off:	<ol style="list-style-type: none"> 1. Check that the chair loop is connected. 2. If the LOOP OK is off even though the chair loop is connected correctly, the chair loop is defective. See "Repairs" below.
The sound does not correspond to the TV picture:	<ol style="list-style-type: none"> 1. If the amplifier is connected in series with several external units, (e.g. video, DVD, satellite box) via the SCART input, switch off the other external units. 2. If you have cable TV with a digibox but are watching an analogue channel, switch off the digibox. 3. If you are watching a DVD film and have the right picture but the wrong sound, connect the Line-Out cord from the amplifier's LINE/DIGITAL output to the DVD player's Line-Out output. 4. If you are recording on the video and watching another TV channel, use connection option D.
No sound is heard when a video camera is connected to the TV for playback.	If you are using connection option C (SCART cable), try out connection option A, B or D instead.
The sound precedes the image.	Adjust the DELAY knob until the sound is synchronized.

Cleaning and maintenance

Vacuum outside of amplifier and chair loop cushion using a soft brush. Use a damp cloth to wipe the outside of the amplifier. Never use cleaning products or solvents. From time to time check that the cords and connections are intact.

Repairs

If your product malfunctions, it must be repaired by a qualified technician. Do not attempt to open the product, since this would invalidate the warranty. (You may, however, remove the grey fabric cover from the chair loop). If your product requires service, please contact your hearing care professional for assistance.

Waste disposal

This product contains electrical or electronic equipment and should be disposed of carefully in the interests of your safety and the environment. Please contact your local hearing care professional concerning disposal of the product.

General security precautions

- During the installation: always start with the volume control set to minimum and then gradually increase the volume to a comfortable level. Hearing may be damaged if the sound level is too high.
- Chair loop: If you know that you have extremely sensitive skin, it may be good to avoid prolonged skin contact with the chair loop. Place the chair loop under the seat cushion or wear, for instance, trousers to avoid direct skin contact with the chair loop while sitting on it.
- Low-energy radiation (e.g. from burglar alarms, room surveillance systems, radio equipment and mobile phones) is not harmful to the product. These types of radiation could, however, momentarily affect the sound quality or create noise in the device.
- Warning: External devices connected to the product must be safe according the requirements of EN 60601-1-1, EN 60335-1 or EN 60065, as appropriate.
- Use only the power adapter which is shipped with the device.
- Protect cables from any potential source of damage.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- If a serious incident occurs in relation to this device, contact the manufacturer.
- Do not drop your device. Dropping onto a hard surface can damage it.
- Do not use the device in areas where electronic equipment is prohibited.

Compliance information

This device is compliant with the following regulations and directives:

- Medical Device Regulation, MDR
- RoHS Directive
- WEEE Directive

Hereby AB Transistor declares that this product, within Europe, is in compliance with the essential requirements of the Medical Device Regulation EU 2017/745. Declaration of conformity can be obtained from AB Transistor.

Part numbers and accessories

AutoMini Digital Set, EU300 00 06, Included in AutoMini Digital, EU:

TOSLINK - miniTOSLINK cabel	300 00 52
Mains adapter	200 10 37
RCA cable with 3.5 mm jack, 1.5 m	200 10 33
Audio cable 3,5 mm - 3,5 mm, 1,5 m	300 01 17
Chair loop	200 20 45

Accessories:

Neck loop with volume control	18 09 04 00
Connection cord for neck loop	200 20 26
FXT microphone with 3.5 mm jack, 4 m	200 60 13
Scart cable	200 10 39
Coaxial digital, SPDIF, RCA - 3.5mm, 1m	300 01 96

Technical data

AutoMini amplifier

Analog input:	>50mV
Digital input:	PCM uncompressed audio over S/PDIF
Output load:	4 Ohm
Power output:	10 W at 4 Ohm
Transient state time:	50 ms
Decay time:	35 s
Frequency range:	80 Hz - 8 kHz, -3 dB rel. 1 kHz
Distortion:	<1%
Signal/noise ratio:	>40dB A-weighted
Delay of sound:	Off and 50-180 ms
Dimensions:	145 x 70 x 30 mm (with attachment approx. 38 mm)
Weight:	150 g (without mains adapter)

External mains adapter:

Plug-in adapter, primary 230V 50Hz, secondary 18V DC, 0,8A

Chair loop:

Resistance:	4 ohms
Dimensions:	41 x 41 x 0.2 cm
Connector:	2-pin DIN loudspeaker plug
Flex length:	10 meters

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AB Transistor Sweden