

# DATASHEET PROTECTIVE PLUS

Advanced Cleaning Solutions for  
Enhanced protection and flexibility in  
Medical Instruments

## CHEMICAL COMPOSITION

Potassium hydroxide, benzalkonium chloride, anti-corrosive agents, deionized water

## PRODUCT SPECIFICATIONS

Protective Plus provides preventive maintenance and sterilization of medical instruments. A first-class medical white oil forms a protective layer that shields instruments from air and moisture, reducing corrosion and allowing smooth joint movement. Leaves no residue on equipment.

## AREAS OF USE

Protective Plus is compatible with a range of materials, including metals, plastics, rubber, glass, wood, porcelain, and stainless steel, making it versatile for various medical and dental instruments.

## USE

Dilute Protective Plus to 2% for use in automatic washers or manually (e.g., 100 ml in 5 L of water). Submerge instruments fully in the emulsion for optimal protection. After washing, allow instruments to air dry; they can be sterilized immediately if needed. Replace the solution when visibly contaminated, it is enough for treated instruments to be disinfected before reuse.

## ANTI-MICROBIAL CHARACTERISTICS

Bactericide, Fungicide, Virucide, Antimycobacterial, Tuberculocide

