

PRODUCT SPECIFICATIONS

It removes the protein residues, iron oxide and other oxidation stains on the instruments. It's material compatibility is high. It is used in manual and ultrasonic washing devices and to prevent the stain formed due to sterilization vapor.

AREAS OF USE

Strong A Plus rust and corrosion inhibitor removes the stains, corrosion and rust on medical devices. It can be used in several fields such as surgery, anesthetics, laboratory equipment, dentistry and etc.

USE

Strong A Plus Rust and corrosion inhibitor is used by dilution with water in any quality (e.g. at 1% concentration; 10 mL of solution in 1 L of water) depending on the rust state of the surface to be cleaned. The product that is diluted in appropriate ratio is filled inside a container, the rusted instruments are immersed into the container so as all the surfaces will contact with the solution, they are taken out of the solution after waiting for minimum 5 and maximum 10 minutes and then rinsed with plenty of water.

+970562600007

www.activemed.ps

