

# Power of **Fentanyl** Testing is in your hands!

*First handheld CLIA Waived & OTC Approved FIA Reader*

# VivaChek

BioSieve™ / Toxismart™ / SalTox™

Ultra Sensitive  
**1 ng/mL**



VivaChek Biotech (Hangzhou) Co., Ltd.  
Level 2, Block 2, 146 East Chaofeng Rd., Yuhang Economy  
Development Zone, Hangzhou, 311100, Zhejiang, P.R. China  
VivaChek Diagnostics, Inc.  
1401 21st St STE R Sacramento, California, 95811  
United States of America  
TEL: (888) 860-3894  
E-mail: [info@vivachek.com](mailto:info@vivachek.com)  
Website: <https://vivachekdx.com>

Number: 1606017203  
Effective Date: 2024-10-07  
©2024 VivaChek Biotech (Hangzhou) Co., Ltd.

# VivaChek

# 01 FIA TESTING SYSTEM

## Product information

Product	BioSieve™ Fentanyl FIA Pro Test Kit
Specimen	Urine
Reaction time	5 min
Cut-off level	1.0 ng/mL

Product	BioSieve™ ToxiSmart FIA Reader
Size	124.5 × 72.5 × 40 mm
Weight	165 g
Connectivity	Bluetooth
Memory	999 results



## Ordering information

Item code (REF)	Description	Size
VCUS-FIARDR	BioSieve™ ToxiSmart™ Reader and BioSieve™ ToxiSmart™ Bluetooth printer with charging cables, SQC, User's Manual and QRI	1 Reader/Box
VCUS-FIARDRPROMO	BioSieve™ ToxiSmart™ Reader and BioSieve™ ToxiSmart™ Bluetooth printer with manuals, SQC test and charging cables and 100 x BioSieve™ Fentanyl FIA Pro Tests	1 Promo/Box
VC-FYL1	BioSieve™ Fentanyl FIA Pro Test Kit, each box contains 25 Tests, 25 Disposable pipettes and Package Insert and QRI	25 Tests/Kit
VC-FYCTL	BioSieve™ Fentanyl Controls, 5 ml Negative and 5 ml Positive Controls (1 Set)	1 Set/Box
VC-TPPR10	BioSieve™ Thermal Paper Rolls for use with BioSieve™ Bluetooth Thermal Printer	10 Rolls/Box

## Fentanyl Test Procedure

- Collect the urine specimen.
- Press the right button "▶" for 5 seconds to turn on the Reader.
- Remove the Test Device and Disposable Pipette out of the foil pouch and place it on a clean, dust-free and flat surface.
- Apply **3 drops (~75 µL)** of urine vertically to the sample well of the Test Device using the Disposable Pipette. Insert the Test Device into the slot of the Reader immediately.
- Please ensure that orientation of the Test Device is as shown in the image above.
- The Reader will automatically display the result after **5 minutes** countdown.

# 02 OTHER LATERAL FLOW RAPID TEST

## Product information

Drug (Identifier)	Cut-off level (ng/mL)	Drug (Identifier)	Cut-off level (ng/mL)
Amphetamine (AMP)	1000 or 500	Morphine (MOP300/OPI2000)	2000 or 300
Secobarbital (BAR)	300	Methadone (MTD)	300
Buprenorphine (BUP)	10	Oxycodone (OXY)	100
Oxazepam (BZO)	300	Phencyclidine (PCP)	25
Cocaine (COC)	300 or 150	Propoxyphene (PPX)	300
2-ethylidene-1,5-dimethyl-3,3-diphenylpyrrolidine (EDDP)	300	Nortriptyline (TCA)	1000
Methylenedioxyamphetamine (MDMA)	500	Marijuana (THC)	50
Methamphetamine (MET)	1000 or 500		

