**REVIVRE**

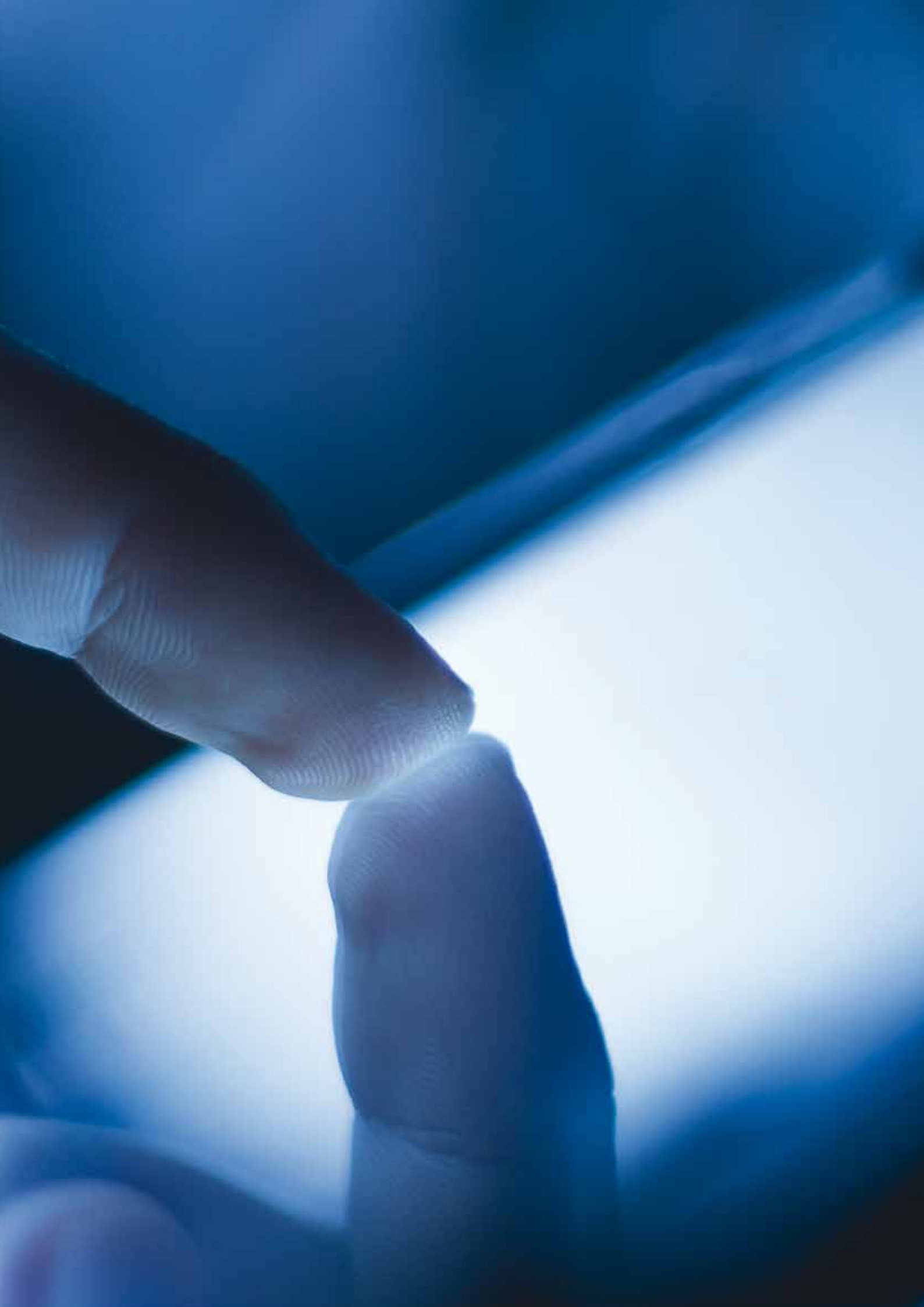
SINERGY

advanced technological synergies

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0.1 synergies

The underlying idea is the availability of several applications using one single equipment.

Our research centres have developed technological synergies to make the beautician's task so much easier, because the beautician is at the centre of the Revivre world. Experience in the science of beauty, with some help from the world of engineers, means we can now create special ways of working to make all our products even better than before. The result of working out the exact molecular weight we need, the texture and consistency of our products, and the absolutely most brand spanking new software is that we can send you some extremely high performance treatments. The Revivre method brings together technical nous and jaw dropping cosmetics.

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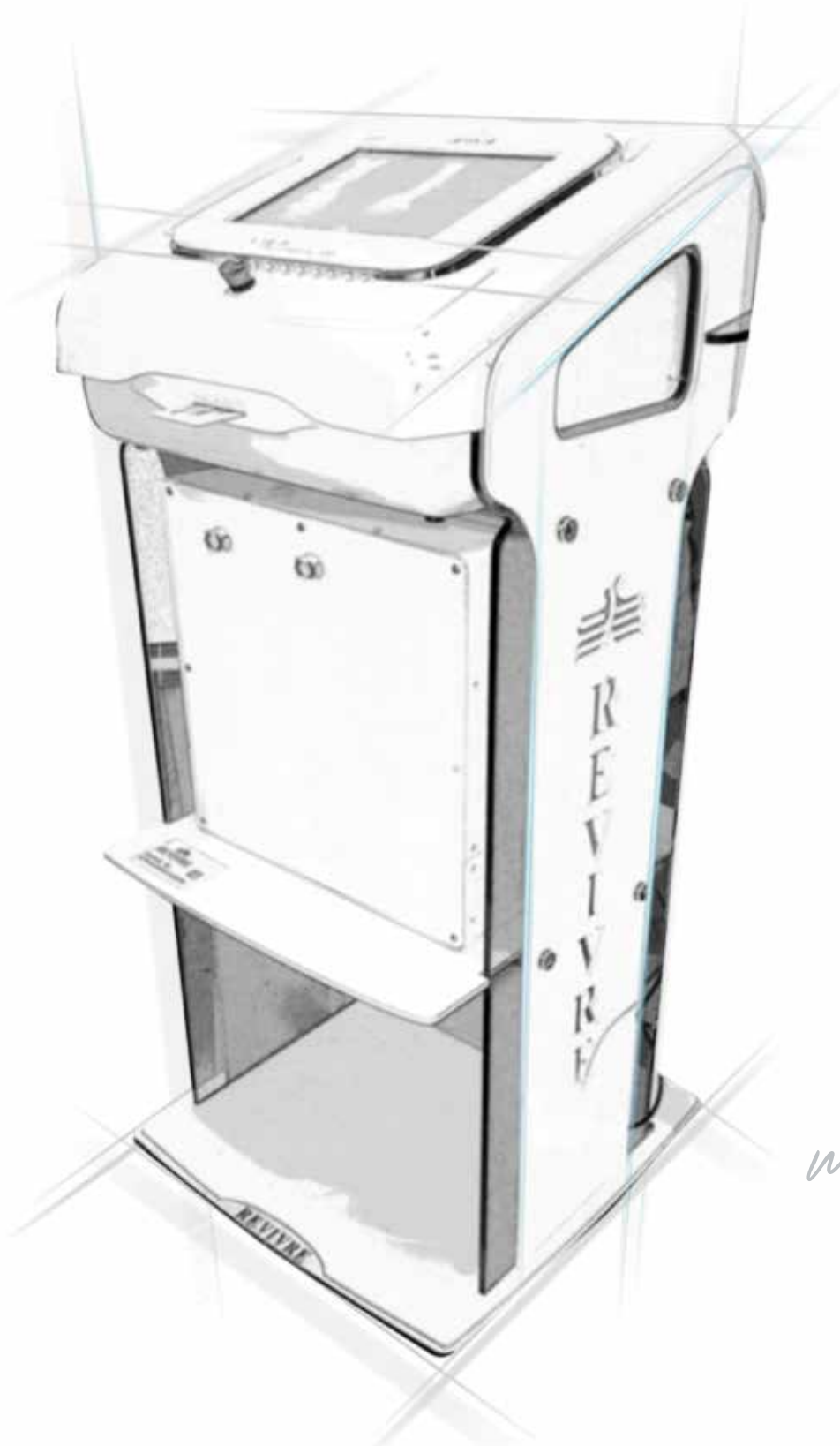
0.2 the importance of design

We have taken care of every single detail so we aren't just doing a job, and so we can turn the tools of our trade into something efficient and easy to use. Oh, and we don't want to forget about performance either.

*All this synergy is **made in Italy**. They are assembled on a chassis we think looks cool, funky and groovy, and which Revivre had specially designed. The 10" touch screen and the accessory shelf make it something absolutely practical in day-to-day use, and turn every piece of equipment into boutique furniture.*

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0.3 research and training

Aware of the education's importance, Revivre has developed a detailed and professional training program.

The customer service is crucial as well as the expertise, technical training and further training. With this optical, the constant updating plays a major role.

Thanks to training programs created by Revivre, you can delve into the technical characteristics of the products and equipment, know the application fields of different technologies, learn the correct use of methods and know how to best advise customers on the basis of blemishes to treat. The courses also represent an opportunity to discover synergies between technology and product, in such a way as to emphasize to extremely the treatment effectiveness and duration.

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Following years of research, in 2010 it opened the Revivre Technical Division, the company division focused on Italian designed and manufactured facial, body and hair removal equipment and technology. Stringent safety, efficiency and durability tests meet both European and American standards.

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0.4 certifications

Revivre Italia SpA is proud to be one of the companies who obtained the quality certification in compliance with UNI EN ISO 9001:2008 and UNI EN ISO 22716:2008

Thanks to this quality certification, Revivre grants its clients and suppliers the highest quality and safety levels both for products and services. Our mission is to sustain a high quality level of our products and services and increase every day the customer satisfaction.

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EXTRAORDINARY FAST, SAFE and PAINLESS METHOD
for unwanted hair removal 365 days a year*

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Photobiostimulation technology comes from a research that has demonstrated that different lights can stimulate more active growth of fibroblasts. A method with immediate results in total safety.

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Light therapy results in the application of energy light from which a natural photochemical reaction is generated, PHOTOBIOSTIMULATION, which has rejuvenating and antioxidant power.

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Adatto a donne e uomini
Indicated for both women and men

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*application 7/10 days prior to sunbathing is not recommended

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14 trattamenti specifici
in abbinamento a R-Shape e Maris Spa
14 specific treatments combined with R-Shape and Maris Spa

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Programma Cellulite
Cellulite Program

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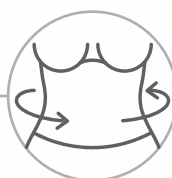
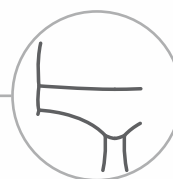
Press11 - Massage is a treatment technology by presso Massage, specially tested by the Revivre laboratories to obtain a draining effect.

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The whole body is treated through an inflation system managed by a microprocessor. The supplied programs are able to satisfy all the needs of beauty, well-being and slimming.

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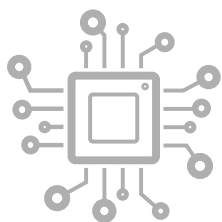
A special microprocessor allows you to vary the amount of air and the pressure within the sectors, in order to create a compression physiological peristatic that evenly envelops the limb from the bottom up the top. This helps blood and lymphatic to outflow and the reabsorption of liquids.

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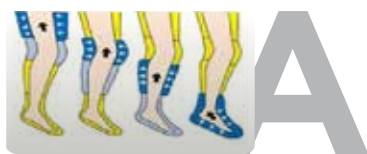
The liquids reabsorbed in the vessels carry the waste retained in the tissues, which according to the their size ends up in the blood system or the lymphatic system.

The compression obtained reduces the space between the adipose system and that circulatory one, facilitating the contact of adipose cells with circulating substances that help to metabolize fats.

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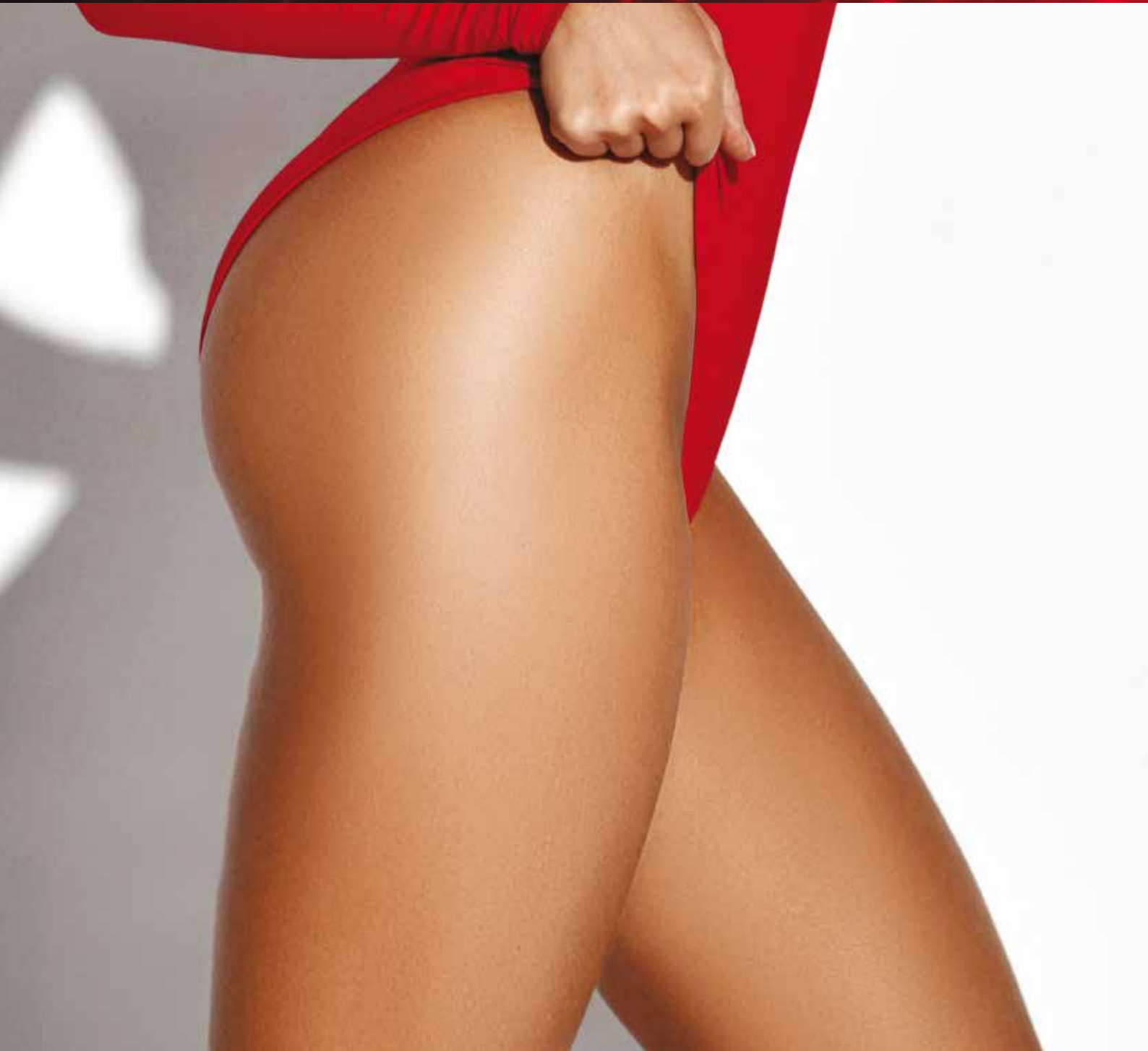
Infrared rays have a direct action on the structure of the fat, releasing great energy thermal. The result is an acceleration of blood circulation, stimulation of the metabolism the acceleration of biochemical reactions, the increase in catabolism, the reduction of adipocytes.

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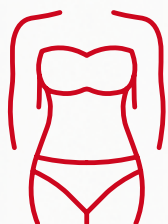
Press11IR - Massage has channels already divided between all areas of the body with the possibility of choosing the single zone or the complete treatment. Thanks to the temperature management on several intensity levels you can choose the functionality of the treatment: sweating, increased metabolism, micro-cycle reactivation and relaxing effect.

It reduces the amount of body fat and especially localized fat. It improves the metabolism, the body becomes more efficient at consuming calories and fat. It therefore improves the aesthetic appearance and capacity of the muscles to exploit fat as an energy source.

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Infrared rays have a direct action on the structure of the fat, releasing a great deal of thermal energy. It follows an acceleration of blood circulation, stimulation of metabolism, acceleration of biochemical reactions, the increase in catabolism, the reduction of adipocytes. The air pockets are inflated and regularly deflate with adjustable pressure, favoring the dissociation and crushing of adipocytes.

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The thermal energy generated by the infrared will decrease and relax the muscle.

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Press1IREL - Focused electrostimulation allows you to awaken unused muscle groups for a long time (or used little) causing laxity with a cascade effect. It allows you to act on the vascular system stimulating lymphatic circulation and opposing cellulite by reducing adipocytes.

Through the contraction of the muscles, a series of positive effects on the body are obtained, which can act in synergy with the pressomassage and infrared techniques. The areas that are usually subjected to electrostimulation are those most difficult to model through nutrition and physical activity:

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NEW RLX WARM is a technology for localised heat treatment aimed at reducing fatty tissues, modelling the figure and restoring the desired form.

The properties of infrared rays have been known for a long time, as they are widely used in physical rehabilitation, to combat muscle contraction and pain and prepare the muscle for physiotherapy. Infrared rays directed onto the area to be treated deep heat the tissues, dilating the blood vessels in the areas treated to promote better local blood circulation therefore better oxygenation of the tissues.

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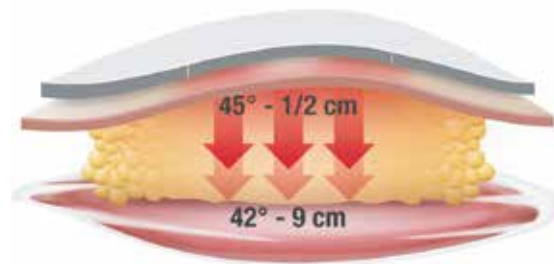
Infrared rays are used in beauty treatments in combination with other treatments to combat “sagging” of the skin. The heat released by the rays stimulates production of new collagen, with beneficial effects on the skin’s elasticity and tone local blood circulation therefore better oxygenation of the tissues.

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We recommend use of products in R-SHAPE PHC and DREN lines for professional treatments, as they have been designed in our laboratories for targeted, personalised treatment protocols. permette di memorizzare il protocollo trattamenti concordato con la cliente.

THE PROGRAMS

PROGRAMME 1: Compact cellulite

Helps resolve the cellulite process known as oedema

PROGRAMME 2: Oedematose cellulite

Works on fat cells to reduce hyperplasia and hypertrophy

PROGRAMME 3: Soft cellulite

Reduces micro-nodules

PROGRAMME 4: Slender legs

4 phases with different temperatures to obtain improved haemato-lymphatic circulation and reduction of fat in the area treated

PROGRAMME 5: Slender arms

4 phases with different temperatures to obtain improved haemato-lymphatic circulation and reduction of fat in the area treated

PROGRAMME 6: Capillary treatment

Specifically for treatment of varicose veins. The temperature remains unvaried in the 4 work phases at points 1 and 2 and changes in the 4 phases on the buttocks and abdomen

PROGRAMME 7: Slender buttocks and thighs

Divided into 4 phases, each with a different temperature, to obtain improved haemato-lymphatic circulation and reduction of fat in the area treated

PROGRAMME 8: Slight blood vessel dilation

For use prior to manual treatments to improve their efficacy and aid penetration of beauty treatments

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■ DIAMAGNETISM FOR FACE AND BODY TREATMENTS

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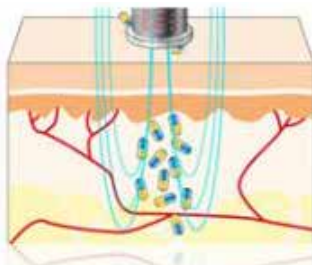
Skin Booster uses the diamagnetism technique, a non-invasive technique that builds its own operating principles on the use of high intensity hyperpulsed magnetic fields.

Almost all molecules (water, vegetable oils, organic substances) are diamagnetic, that is, they tend to move away from an applied magnetic field. Generating a pulsed magnetic field of opportune frequency on the skin, this then sets these substances in motion, present both inside and outside the cells.

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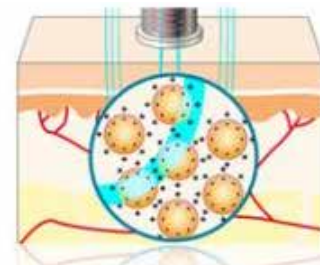
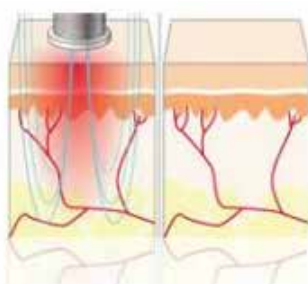
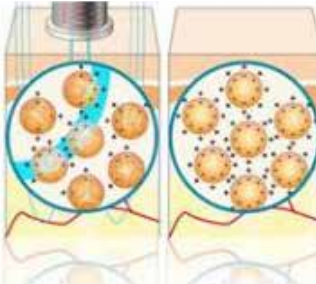
Biostimolazione endogena
Endogenous biostimulation

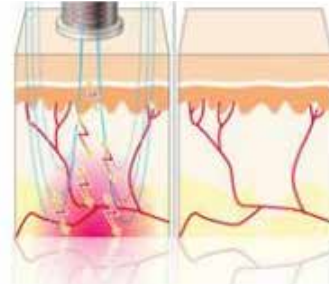
Medicina
Medicine

Accellera i processi
riparativi tissutali
Speed up processes
tissue reparatives

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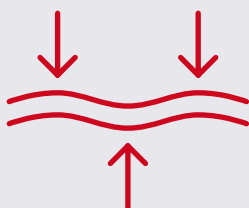
*DEEP ABSORPTION
OF ACTIVE INGREDIENTS*

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The absorption of active molecules occurs in a **NON-invasive** way (no electric currents!), but exclusively by mechanical thrust.

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Improvement of the penetration of active cosmetic applied during treatments.

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Endogenous biostimulation is capable of promoting development of electric currents on the level of different fabrics, stimulating them natural activity.

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The **repulsion exerted by the field magnetic towards water body** promotes movement liquids.

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Drainage: calling back, from the layers deep towards the more superficial ones, the blood and lymph containing waste, toxins and metabolic residues which in such way are more easily expelled by the body.

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V-Shape is a multi-function technology, in able to carry out both body and face treatments, with single or simultaneous programs, matching more actions at the same time.

It is an endodermal treatment, which allows to treat all the alterations of the connective tissue through the latest generation technologies in the aesthetic field, expressly studied to reduce the imperfections of cellulite and the localized adiposity following a demolition of the calcified fat, that allows to minimize stretch marks, reduce wrinkles, bags, firm and tone dull tissues, for example: under the chin, oval face, abdomen and hips, upper and lower limbs, making a notable remodeling.

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The 850 nm infrared laser reduces skin impedance by heating the skin and the RF energy penetrates deeply into the connective tissues.

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Innovative technology for NON-INVASIVE face and body lifting. Works with a double NATURAL synergical effect of heat and vibration (VT source).

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The selective vibration penetrates deep into the skeletal muscle receptors responsible for control of muscle contraction, regulating its activity and fixing it in their working memory.

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Controlled thermal penetrates deeply into the tissues and reactivates cell metabolism and microcirculation.

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aumento del benessere, relax
increased well-being, relaxation

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NON-INVASIVE FACE AND BODY LIFTING

Guarantee of IMMEDIATE and LASTING results

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Radio frequency is an instrument that causes an increase in internal temperature of the organism (40/45°) causing the phenomenon of "hyperemia".

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Electroporation consists in a sudden opening of the membranes allowing for greater permeation.

COS'È SOFT LASER & LIPO LASER / WHAT IS SOFT LASER & LIPO LASER

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They are led diodes able to stimulate the metabolism, amplifying the effects of the radio frequency and skin oxygenation.

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SOFT LASER IR

corona centrale 10 led 770 nm
corona esterna 12 led 860 nm per
trattamenti viso anti age, ossigenazione tissutale,
stimolazione elastina
*central ring 10 led 770 nm
external ring 12 led 860 nm for
anti-aging face treatments, tissue oxygenation, elastin
stimulation*

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BODY & FACE RF is a technology for performing beauty treatments on the face and body with radiofrequencies.

Radiofrequency raises the body's internal temperature (40/45°), causing “hyperaemia”. This results in increased blood flow, improvement of lymph circulation, the contraction of superficial and deep collagen fibers, stimulation of tissue repair processes, faster local metabolism, better oxygenation of tissues.

FOR A BLEMISH-FREE BODY AND AN AGELESS FACE

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Body & Face RF is a capacitive monopolar radiofrequency with an active effect on soft tissues with a high moisture content (muscles, vascular-lymphatic system) which ensures greater effectiveness by deep penetrating the dermis as far as the outer layer of muscle, passing through the fibrous septa of the hypodermis.

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For long-lasting results and a face that continues to attract attention, use Revivre home care products to preserve the benefits of professional treatment.

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***FOR A FLAWLESS BODY
AND AN AGELESS FACE***

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SENO BREAST

- ✓ Trattamento tonificante / *Tonifying treatment*

CORPO BODY

- ✓ Trattamento tonificante / *Tonifying treatment*
- ✓ Modellamento silhouette / *Modelling of the figure*

PIASTRA
PLATE

SET DISCHI
RADIOFREQUENZA
SET OF RADIOFREQUENCY
DISKS

MANIPOLO
HANDPIECE

**GETS RID OF WRINKLES
WITH AN INSTANT LIFTING EFFECT**

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WHAT IS

It is a hi-tech professional device for face skin check-up.

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HOW TO USE IT

Practical, easy to handle and intuitive, after the initial settings and data entry of the customer (age, gender, etc.), the device is ready for advanced professional check-up of the skin. Just follow the instructions that appear on the screen, place the micro camera on specific areas of the face and photograph the selected area.

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THE SKIN CHECK UP

Digital Pro Skin Check allows you to make a complete and very careful diagnosis of cutaneous imperfections. **During the consecutive phases of the check up, are evaluated:**

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After each phase of the check up, the photograph of the affected area is compared with 3 different reference images that indicate the different levels of the imperfection. At the conclusion of all measurements, a special graph displays very clearly the final result of the test.

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THE “RECIPE” FOR THE CUSTOMER AND THE SENDING OF THE RESULT

Advising the costumer on the most appropriate treatment path is simple and immediate. At the end of the check-up, the personalized “recipe” indicating the treatments to be carried out in the institute and the products for the home beauty routine. To verify the improvement of the treated imperfections, all the data collected can be saved in the device memory and sent to the customer via email.

VISO LUMINOSO E TONICO
RADIANT AND TONED FACE

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Ever attentive to customers' skincare needs, Revivre research laboratories have perfected a technology for the treatment of the face, neck and décolleté:

✓ Peeling Meccanico
Mechanical peeling

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already after the first applications
SKIN APPEARS MORE TONED AND COMPACT!

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**Deluxe treatment for a
RADIANT, TONIC COMPLEXION**

**MICRODERMOABRASIONE
*MICRODERMABRASION***

VISO LUMINOSO E TONICO
RADIANT AND TONED FACE

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Mechanical peeling, also known as “microdermoabrasion”, is a painless non-invasive treatment that improves the appearance of facial skin where ageing, sunshine and acne have produced “flaws”. Rubbing the outer layer of the dermis with the diamond-tipped head has a deep exfoliating action, so that after completion of treatment the skin will be smoother, cleaner, fresher and above all renewed!

Diamond Face, Revivre’s mechanical peeling treatment, comes with three interchangeable heads for treating different types of skin.

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An innovative, highly effective treatment for sagging of the skin on the face, neck and décolleté. Vacuum massage consists of slight aspiration of the skin tissues of the face using a vacuum system in order to reactivate capillary circulation and improve blood supply to the tissues. A handpiece with 2 spheres and a probe handpiece are used to treat difficult, delicate areas such as the face, eye area and neck.

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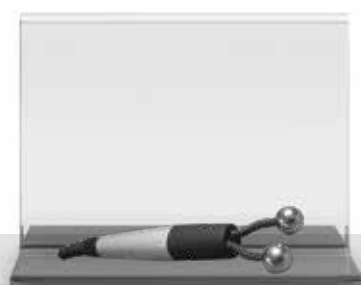
Also known as the “facelift without surgery”, this technique uses transmission of microcurrent at medium frequency to stimulate the muscle to lift and stimulate creation of new collagen. The tissues are regenerated, oxygenated and nourished and the skin is more receptive to the active ingredients in Revivre beauty treatments.

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Electroporation consists in a sudden opening of the membranes allowing for greater permeation.

MANIPOLO A DUE SFERE TWO-SPHERE DEVICE

MANIPOLO CORPO BODY DEVICE

MANIPOLO VISO FACE DEVICE

BARRA MASSA BARRA MASSA

VISO LUMINOSO E TONICO
RADIANT AND TONED FACE

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Riempimento e sollevamento del viso
The skin of the face is filled in and lifted

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OXY Therapy is an OXYGEN concentrator: it collects atmospheric oxygen, purifies and conveys it under pressure through the airbrush directly on the skin. All the power of nature in a device with advanced technology capable of deliver up to 93% pure oxygen and give new life to the tissues that due to different factors undergo stress and fatigue every day.

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OXYGEN is the main element for the life of all cells of our organism.

Energy is obtained from oxygen and is used in all functional processes of the cell. In the past it was supposed that oxygen reached the most distant districts only through blood vessels.

Today, however, it has been scientifically proven that the surface layers of human skin are almost exclusively supplied with oxygen from the external environment.

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Thanks to the work of two internal cylinders, one filled with zeolite, which selectively absorbs nitrogen in the air and another necessary to expel the oxygen collected, oxygen is separated from the air environment, allowing the administration of highly purified oxygen. Through one special airbrush, Oxy uses oxygen as a vehicle for the application of special active ingredients carefully chosen and studied by REVIVRE laboratories.

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The product to be conveyed is dosed by the operator in order to be more precise during treatment. In this way the oxygen supply is increased to the cells of the deep skin layer and the penetration of active ingredients.

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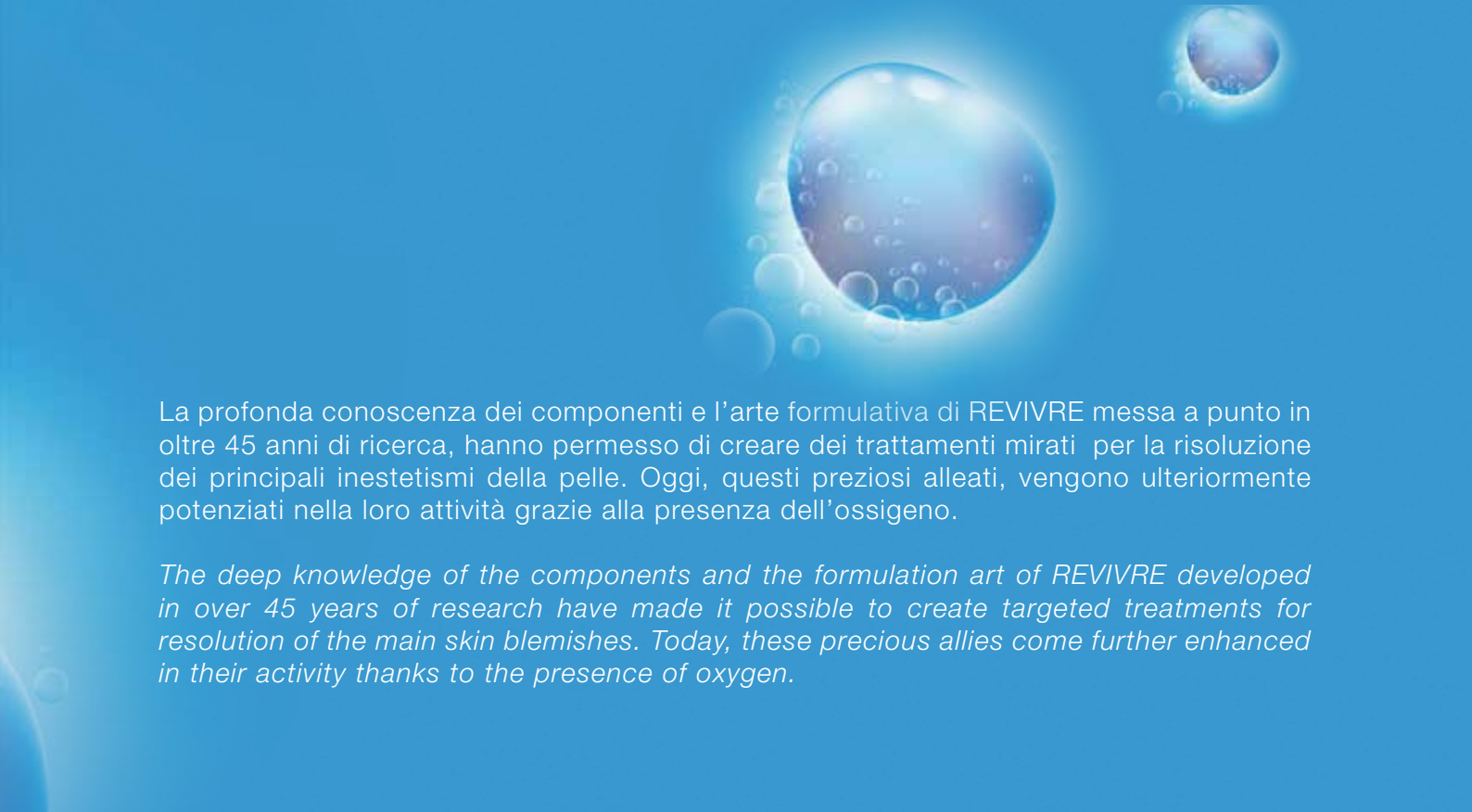
The deep knowledge of the components and the formulation art of REVIVRE developed in over 45 years of research have made it possible to create targeted treatments for resolution of the main skin blemishes. Today, these precious allies come further enhanced in their activity thanks to the presence of oxygen.

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A GLOBAL ANTI-AGING EFFECT

The synergistic multifunctionality of Complete Beauty Face allows you to improve or correct blemishes by choosing from guided programs or specific treatments. Thanks to the preventive analysis with the special macrocam x200 and the handpiece for the hydration control, it is possible to examine the quality and needs of the skin on the areas of the face that require more attention.

Every single program of Complete Beauty Face has been designed to answer to the different needs of the skin, allowing the professional to give to its customers a beauty experience with a global anti-aging effect.

- ✓ Software intuitivo/ Intuitive software
- ✓ Controllo idratazione / Hydration Control
- ✓ Micro dermoabrasione / Micro dermabrasion
- ✓ RF multipolare / Multi-pole RF
- ✓ RF frazionata / Fractional RF
- ✓ Ultrasuoni 3 MHz / 3 MHz ultrasound
- ✓ Mask light face / Mask light face

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HYDRATION CONTROL: allows the detection of parameters in 4 precise points of the face and then performs an average of skin hydration

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MICRO DERMOABRASION: performs skin exfoliation treatments, favoring the natural cell turnover and better absorption of the cosmetics used subsequently.

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RF MULTIPOLARE: helps to treat expression lines and small wrinkles, giving brightness and elasticity to the skin.

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FRACTIONAL RF: new fractional radio frequency technology that allows targeted work on the deepest wrinkles.

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ULTRASOUND 3 MHz: gently warms the skin tissue to promote the physiological metabolism cellular, helps maintain a correct degree of hydration and facilitates the absorption of products cosmetics.

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MASK LIGHT FACE: the chrome frequency considers the imperfection a disharmonious vibration of skin cells, which can be rebalanced thanks to the energetic vibration of one or more colors specific.

TECNOLOGIA MULTI SINERGICA PER RISULTATI VISIBILI
MULTI-SYNERGIC TECHNOLOGY FOR VISIBLE RESULTS

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AN EASY AND IMMEDIATE INTUITIVE SOFTWARE

The Complete Beauty Face monitor immediately becomes an essential tool to implement the professional's work. With a few small gestures, the operator is able to provide 360 ° advice on the specific situation of the customer.

It is possible to consult images, programs, treatments, data of customers and the situation of the treatment path and consequently exploit the emotional message to involve the client in the course of treatment.

Revivre is committed to creating an eco-friendly corporate environment as well as the use of emerging green technologies in the production of its extensive line of environmentally friendly products.

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REVIVRE ITALIA S.p.A - Milano - Italy
Tel. +39 0424 8818 - Fax +39 0424 881888 - info@revivre.it
www.revivre.it