

# Surgical Marking Pen

## Product Performance and Working Principle

This product is a device used to mark on the patient's skin, such as outlining an incision site or identifying anatomical landmarks for blood pressure measurement. It can be widely used in general surgery, plastic surgery, orthopedics, neurosurgery, dermatology, radiation therapy, and other related fields requiring surgical marking.

Model	Specifications	Ink Color
SMD-SP-01	0.5mm	Purple/White
SMD-SP-02	1.0mm	Purple/White
SMD-SP-03	0.5mm + 1.0mm	Purple/White




**Packaging:** 100pcs/bag ; 200pcs/box ; 800pcs/ctn

\*Other sizes can be customized according to customer requirements

## Instructions for Use

1. Clean the skin area to be marked and allow it to dry.
2. Use the surgical marker to make marks on the skin.
3. To remove the marks, wipe them off with low-concentration alcohol.

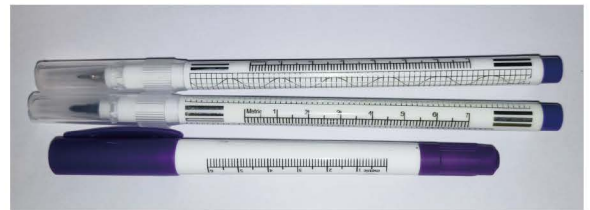

## Warnings and Precautions

1. Each marker is for single-patient use only to prevent cross-infection.
2. Protect the tip during use; cover the cap when not in use to prevent the ink from drying out.
3. Do not use if the packaging is damaged before use.
4. Use with caution on mucous membranes, broken skin, or open wounds.

## Storage Instructions

Store in a dry, well-ventilated environment free from corrosive gases, and avoid direct sunlight.

## Shelf Life

3 years from the date of manufacture.



Website: [www.sainsmed.com](http://www.sainsmed.com)

E-mail: [sales@sainsmed.com](mailto:sales@sainsmed.com)

Manufacturer: Hubei Engi Biotechnology Co.,Ltd.

Phone Number: +86-15295546603

What's up: +86-15295546603