The Durex logo is a blue oval with the word "durex" in white lowercase letters. Below it is a circular seal with the word "durex" in the center, "QUALITY" at the top, and "QUALITY" at the bottom.

1 Handle the condom carefully, as it can be damaged by fingernails and sharp objects.

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EN With the roll outside, squeeze the teat and place the condom on the top of the erect penis.

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EN Roll down to its base. Stop and check if you feel the condom slipping off or tightening excessively onto the penis because this might lead to breakage. Soon after ejaculation the condom should be taken off. The condom should be held firmly in place at the base of the penis before withdrawal.

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DUREX and the DUREX logo are Trade Marks of The Reckitt Benckiser Group of Companies. Discover us online at: www.durex.com UK Carline 0333 2005 345, ROI 01 630 5429

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Reckitt Benckiser Healthcare (UK) Limited, Ransom Lane, Hull, HU8 7DS, UK

EC REP RBL Brands B.V. Schiphol Blvd 207, 1118 BH Schiphol, NL

Service Consommateurs France: 0033 (0) 69 93 19 50

0032 (0) 2456 9701

0031 (0) 23558 4501

Reckitt Benckiser, Sydney, NSW, Australia

Reckitt Benckiser Auckland, New Zealand

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Trk. 210 8127253

Store in a cool dry place away from direct sunlight.

ISO4074:2015 and associated quality marks apply to Durex latex condoms only

ISO23409:2011 applies to the Durex Polysoprene (synthetic latex) condoms only

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Reckitt Benckiser Deutschland GmbH, Darwinstrasse 2-4, D-69115 Heidelberg
Reckitt Benckiser Austria GmbH, Guglgasse 15, A-1110 Wien
Reckitt Benckiser (Switzerland) AG, Richtstrasse 5, CH-8304 Wallisellen
Reckitt Benckiser (Canada) Inc., Mississauga, Ontario, L4W 5S9
Importato e distribuito da Reckitt Benckiser Healthcare (Italia) S.p.A., Via Spadolini 7, 20141 Milano
Yasal Üretici: RBL Brands B.V. Schiphol Blvd 207, 1118 BH Schiphol, NL